


Infant Scale

Model # NK3000

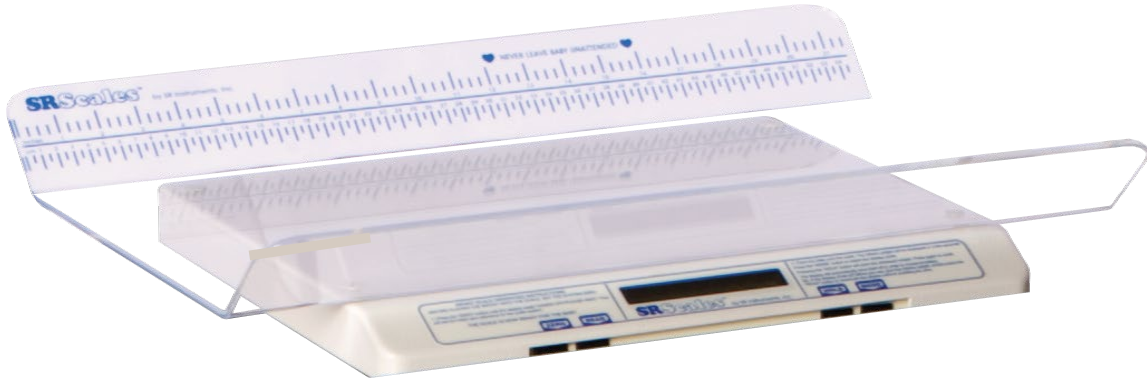

This pediatric scale provides precise and reliable weight information for children up to 40 pounds. The LCD display screen is easy-to-read with various modes available.

Features:

- Portable and light-weight unit
- Recalls the last stored data
- Battery operated unit
- Aluminum plate construction
- Unit durable and easy to clean with plastic shell
- Optional 30" Closed End Tray available
- Optional stainless steel cart for easy transport

Specifications:

- Max Weight: 40 lbs.
- Display: LCD 16 Character
- Accuracy: 0.1% +/- 1 digit of displayed resolution
- Units: pounds/kilograms
- Battery Life: est. 10,000 weigh cycles
- Weight Recall: Press "HOLD" to recall last stored weight
- 4 Modes: Pounds/Ounces (1/4 oz. increment), Pounds Decimal (.01 lb increment), Kilograms (.005 kilo increments), Grams (5 gram increments)

INFANT SCALE OPERATING INSTRUCTIONS
BEFORE PLACING THE INFANT ONTO THE SCALE, SET THE SYSTEM ZERO
1. Press the "ZERO" button until the display reads "HANDS OFF/PLEASE WAIT. This will set the initial zero reference for the scale system.

THE SCALE IS NOW READY FOR THE BABY

ZERO

READ



nk pediatric equipment

a NOVUM Medical Products Company

2. Place the baby onto the scale. The infant's weight will be displayed in a few seconds. Press the "MODE" button to select the display mode. Pressing the "HOLD" button will store the displayed weight. Press again to recall. The display will automatically shut itself off in order to conserve battery life. If the display shuts off before you can view the infant's weight, simply press the "READ" button to reactivate the display without resetting the zero.

HOLD

MODE

Phone | 716.759.7200
Toll Free | 800.274.2742
Fax | 716.759.0700
Email | info@novummed.com
www.novummed.com