



Ultra Sound Chair


PARTS

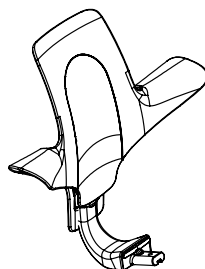


Back (x1)

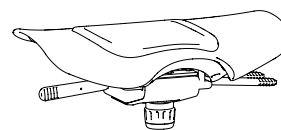

Seat (x1)

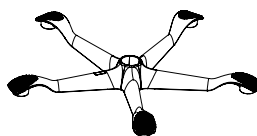

Base (x1)

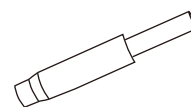

Cylinder (x1)



Casters (x5)

TOOLS



Allen Wrench (x1)
(attached to the bag with casters)

HARDWARE




Bolts (x1)
(attached to the mechanism)



STEP 1

Attach the casters and cylinder to the 5-star base.

1A. Firmly press the stems of 5 casters into the 5 holes in the base.

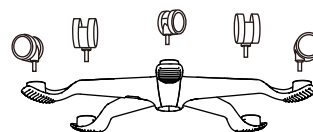


1B. Insert cylinder into the center hole.

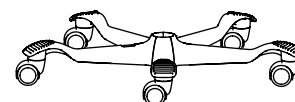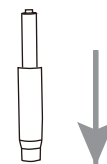


STEP 2



Attach the back to the seat.

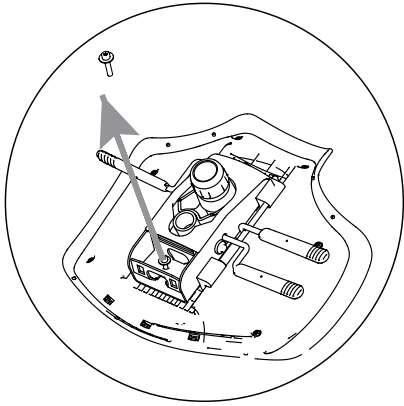

2A. Using the Allen wrench, remove the single bolt from the mechanism.

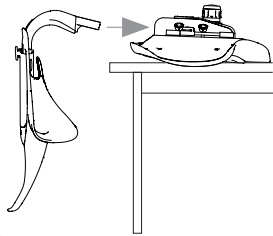

2B. Place the back plate under the seat as shown.

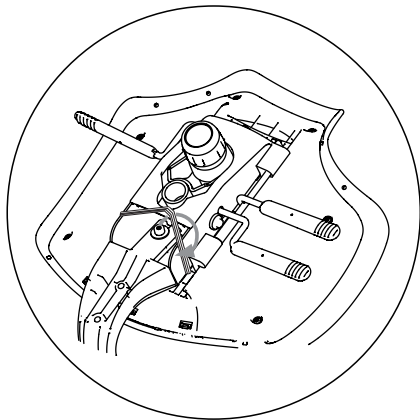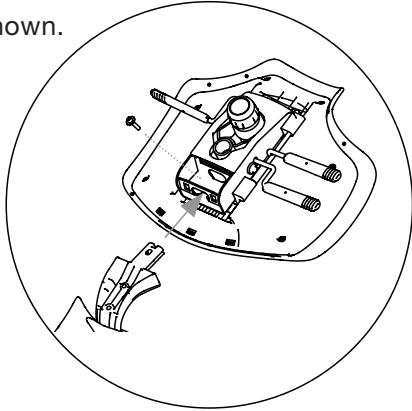


2C. Using the Allen wrench, tighten the bolt from the top.

STEP 3

Join the back and seat with the cylinder and base.

Insert the assembled back/seat unit onto the cylinder, which was placed in the base in Step 1. Press down firmly to secure.

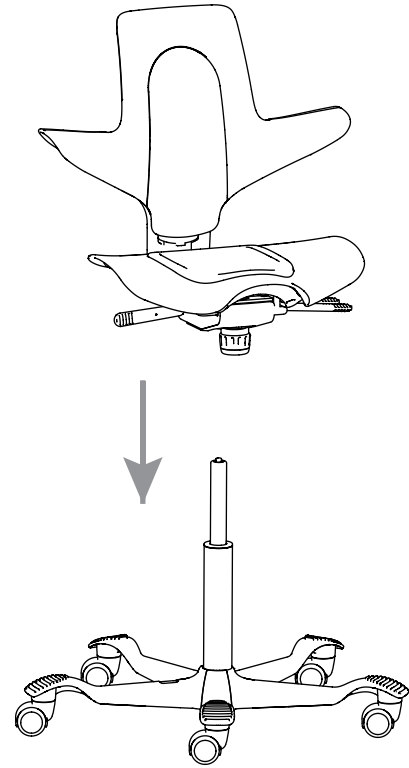

Ultra Sound Chair

PARTS

Missing parts or hardware?
Call 1.800.274.2742



Footring (x1)





Sleeve (x1)




Spacer (x1)

Standard Height Foot Ring

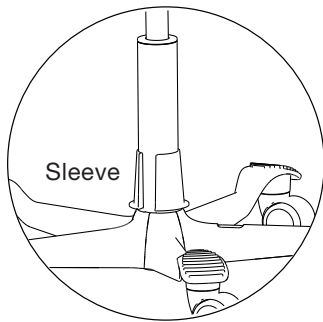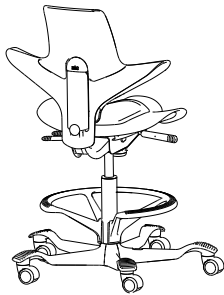


1.A Attach sleeve at narrow part of cylinder

1.B Slide down so sleeve rests at top of base


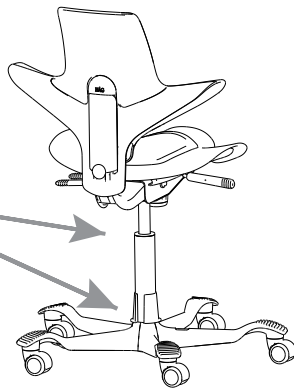


2.B Attach foot ring at narrow part of cylinder

2.B Slide down over sleeve to secure


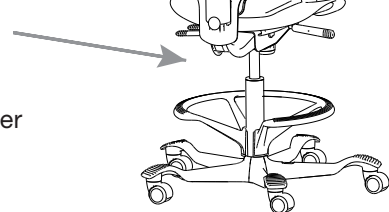


Elevated Height Foot Ring

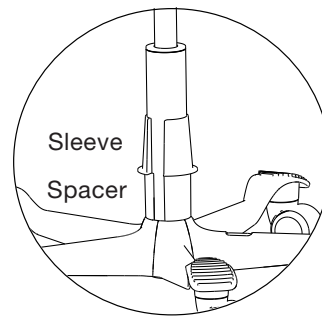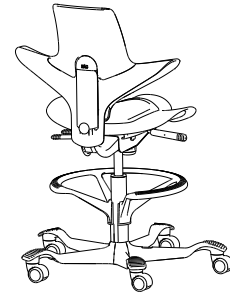


1.A First attach spacer at narrow part of cylinder, slide down so it rests at top of base

1.B Next Attach sleeve at narrow part of cylinder and slide down so it rests on top of spacer

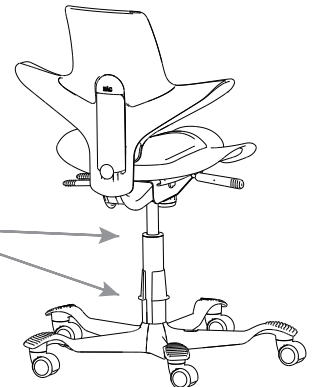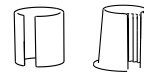

2.B Attach foot ring at narrow part of cylinder

2.B Slide down over sleeve to secure

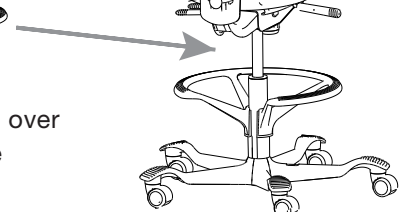



PHONE 1.800.274.2742 FAX 716.759.0700 EMAIL info@novummed.com
www.novummed.com 80 Creekside Drive. Amherst, NY, 14228

©2022 Novum Medical Products LLC